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Art. no. 40002 cool down FIXATION BANDAGE

Art. no. 40003 cool down COTTON BANDAGE 6 x 150 cm

Art. no. 50001 cool down MIXING JAR 400 ml  
(for Art. no. 11001)

Art. no. 50002 cool down MIXING BOX 1800 ml  
(for Art. no. 11002)

## 2 Intended use

For cooling and pain relief in limb injuries, widely applicable.

## 3 Instructions for use\*

In general, the swelling of injuries can be reduced with cooling, rest, elevation and compression.

cool down products are applied locally to the skin and cool (the muscles) by 7 10°C for up to 2 hours. cool down can be used several times a day. cool down contains menthol and camphor, which provide an additional pain-relieving and cooling effect.

Wrap the foot, hand, leg or arm with the liquid-soaked cool down COTTON BANDAGE (Art. no. 40001/40003). cool down FIX AID (Art. no. 20001) can be applied directly. Before use, the cool down ICE FLUID (Art. no. 11001/11002) must be diluted fivefold in the cool down MIXING JAR / MIXING BOX (Art. no. 50001/50002) and the cool down COTTON BANDAGE (Art. 40001/40003) soaked in it. The cool down FIXATION BANDAGE (Art. no. 40002) can be used to fix the cool down COTTON BANDAGE so that the soaked cool down COTTON BANDAGE can also be used under clothing.

*\*Patients are intended users, note for professional users, see chapter 7*

## 4 Indications

Sprains, bruises, blunt trauma and strains

## 5 Contra-indications

Open wounds, eczema

Pregnancy and breast feeding

Infants and young children under 10 kg

## 6 Side effects

In rare cases local skin irritations, contact allergies.

If allergies occur, please contact a doctor or a qualified person.

## 7 Notes for doctors and physiotherapists

For cooling, swelling and pain relief in orthopedics immediately after operations and during rehabilitation.

Fresh scars, abrasions and open wounds must be covered with a foil (Cuarapor, OpSite).

For the cooling and for the muscle regeneration before and after competitive sports.

For pain relief in chronic diseases such as rheumatism and arthritis

## 8 Storage

Store at room temperature, protect from heat

## 9 Disposal

Solution must not be allowed to enter surface water or sewage system.

## 10 Warnings

cool down ICE FLUID is highly flammable. Keep away from heat, hot surfaces, sparks and open flames

## 11 Manufacturer

cool down AG, Gaswerkstrasse 15 CH-8570 Weinfelden TG /  
Tel. +41 71 620 20 66 / Fax +41 71 636 25 43

## 12 Symbols / Meaning

Use before

Lot number

Manufacturer

Catalogue number

Follow the instructions

Upper storage temperature

Multiple use on a single patient

Highly flammable, combustible substances

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